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## EN Instructions

A product for hospitals and maternity homes.

### For babies weighing

Soother for premature babies, Preemie Care (S)	approx. 2,000 g or less
Soother for premature babies, Preemie Care (M)	less than 3,000 g

Even if the baby is within the designated weight range, please consult with a physician before deciding to use the Soother.

### 1. Before use

- Consult with a physician before using the Soother and rely on his or her judgement.
- Before touching the Soother, make sure to wash your hands and fingers thoroughly with soap and clean water.
- When using this Soother, check to make sure that it is not damaged or worn out.
- The purpose of this Soother is to calm babies and to support oral stimulation. It should not be used for any other purpose.

### 2. Cleaning

- Clean and disinfect the Soother before using it for the first time, and after each use. Also sterilize it when necessary.

### 3. Sterilization, Disinfection, and Storage

Choose among the following three types, according to the usage environment.

- \* Do not use sterilization or disinfection methods other than high-pressure steam, chemicals, or boiling, because other methods may cause deterioration of the rubber material, reduced strength, or breakage.

#### ■ Autoclave

- Depending on the instructions for the autoclave, sterilize the Soother either for 15 minutes at 121°C or for 3 minutes at 134°C.

- Using the autoclave under other conditions may lead to insufficient sterilization or cause the Soother to deteriorate rapidly.

- \* Sterilization may cause the silicone rubber to change color. If that happens, replace it early.

#### ■ Chemical method

- Follow the instructions on the disinfectant package to learn how to prepare the solution and how long to soak the Soother.

- \* This disinfection method does not involve heat, so it is less likely to lead to deterioration of the Soother.

- \* Make sure to abide by the warnings on the disinfectant package.

#### ■ Boiling method

- Determine an appropriate boiling time in consideration of factors such as the main kinds of microbes to be controlled, or whether the environment is one in which many babies use the Soother.

- \* Be aware that the Soother may become deformed if it touches the inner surface of the pot.

- \* Make sure to add sufficient water to prevent the pot from boiling dry.

- \* Do not leave a boiling pot unattended during disinfection.

#### ■ Storage

- Dry the Soother after cleaning or disinfection and store it in a clean place out of direct sunlight.

- \* The Soother may deteriorate if exposed to direct sunlight for a long period of time.

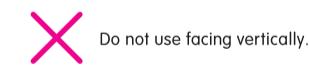
### 4. Cautions

- If the entire Soother fits inside the baby's mouth, it should not be used.
- Make sure that the Soother is not impeding the baby's breath.
- Make sure that the Soother is not covering the baby's nose and interfering with breathing.
- Make sure that the Soother does not come into contact with the baby's eyes.
- Do not tie the Soother around the baby's neck.
- Do not use the Soother for the baby who has begun teething.
- Do not cut the Soother to change its size, or otherwise modify it in any way.
- Do not use the Soother for intubated babies (including both tracheal and gastric intubation).
- The Soother may become discolored due to water absorption.
- If you sense something wrong in cases other than those described above, stop using the Soother.
- Keep an eye on the baby who is using the Soother.

### 5. Guidelines for replacement

It is recommended to replace your Soothers every two months. If you notice any damage such as tears, cuts, or deterioration, replace your Soother immediately.

### 6. Correct use (proper insertion)

Parts	Type of material
Soother	Synthetic rubber (silicone rubber)

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